

Attaching the Accessory Holder to a Visionflex Telehealth Cart


Cart Type	Visionflex Telehealth Cart.
Battery kit	Clear Plastic Holder from Visionflex, Universal Cart Accessory Holder Adaptor Plate, Qty 2 x M4X20mm screws.
Tools Required	Phillips-head screwdriver

1. Ensure that your package contains Clear Plastic Holder from Visionflex, Universal Cart Accessory Holder Adaptor Plate, Qty 2 x M4X20mm screws.



2. Attach the Universal Cart Accessory Holder Adaptor inside the Cart's working cabinet.


*Note: If you have AIO Cart (Picture 1), place the top part of adaptor and push the bottom harder.
If you have Laptop Cart (Picture 2), place the bottom part of the adaptor to hook and push the top lightly.*

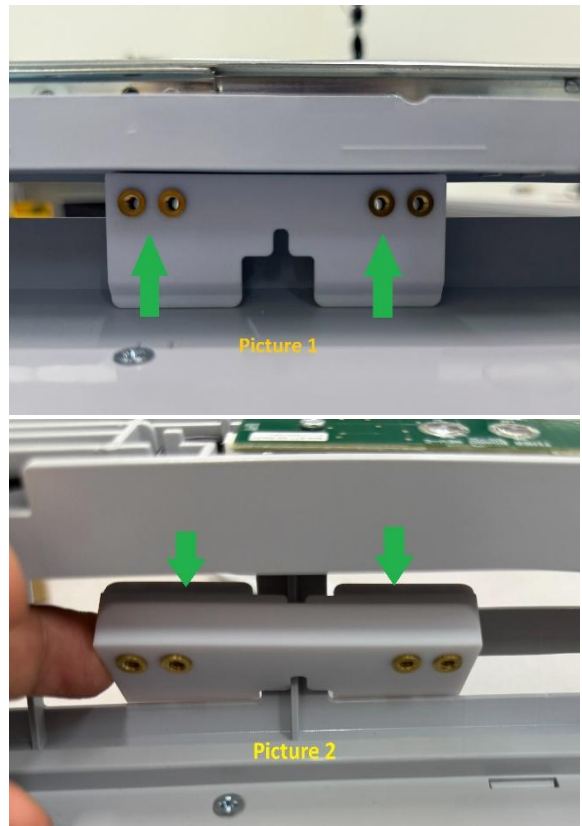

3. Place the Clear Plastic accessory holder with 2 M4x20mm screw from outside and align it with screw holes. Please refer to the picture on the side.

Note: You can select any pair of holes for Laptop cart but highly recommended to use top pair of holes for AIO cart to avoid collisions while sliding the cart's working plate.

4. Use the Phillips-head screwdriver to tighten the M4 screw.

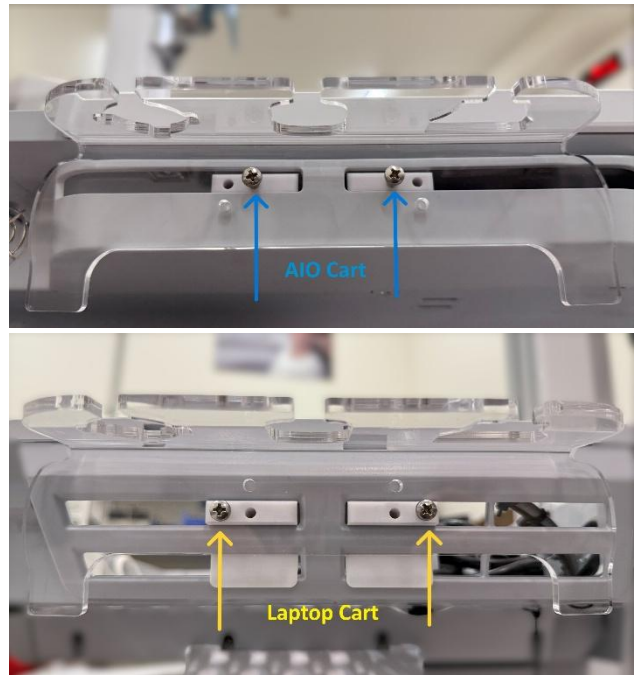

For more guides, training videos and to request assistance visit: www.visionflex.com/downloads-and-support