

Vision Virtual Care Platform

AUDIO SETTINGS PRE-SESSION CHECKLIST


Is the Speakerphone connected?

Double check the connection by removing the USB and firmly plugging it back in.



Is the Speakerphone set as the default Audio Device in Windows?

Right-Click the speaker near the clock to open **Sound Settings** and confirm that the **blue dot** is placed next to Speakerphone for both **Input** and **Output**.

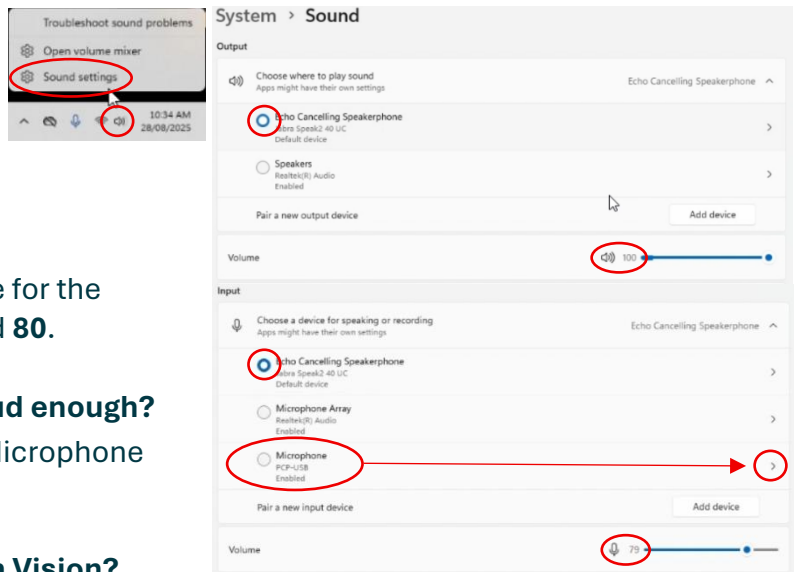

Is the Stethoscope loud enough?

In **Sound Settings** check the volume for the **PCP-USB** device and set it to around **80**.

Is the Speaker and Microphone loud enough?

Check that the Speakerphone and Microphone aren't muted and are loud enough.

Are the correct devices selected in Vision?

Click the **settings button** in the Vision and set both the **Input** and **Output** devices to the **Speakerphone**. **Test** and **Save As Default**.



Are the Headphones charged and turned on?


There is a USB cable to charge the Bluetooth headphones while not in use.

Press the **middle button** for 2 seconds a **blue light** will flash.



Are the headphones set as the listening device in TrueSteth?

Make sure the headphones are switched on first, *then* open **TrueSteth** in Vision, and select the headphones as the **Audio Out** device. Press **Play** to hear the clinical audio.

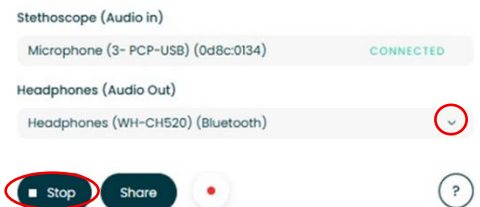

Contact Visionflex technical support
visionflex
After Sales and Technical Support
 Phone: +61 2 8914 4000
 Email: support@visionflex.com

View videos, guides, manuals, log a support ticket or book a training session at visionflex.com/support